

Recommended protocol for CD117

Obtained in run

13 Jul 2016

Immunostainer

Type: Autostainer, LabVision

Primary antibody

Clone: YR145
Producer: Cell Marque
Product no. / lot no.: 117R-14 / BSH1-1zq
Diluent: Antibody diluent
Dilution factor: 1:150
Incubation time / temperature: 30 min. / 20°C

Epitope retrieval, HIER

Device: PT-link / PT-module
Buffer: Std Tris-EDTA / EGTA pH 9
Heating time at max. temp.: 20 min.
Maximum heating temp.: 98°C

Visualization system

Producer: Nordic Biosite
Product / no: NS / KDB-10046
Incubation time polymer: 20 min.
Incubation temperature: 20°C

Chromogen

Producer: Nordic Biosite
Product / no: DAB / BCB-20032
Incubation time / temperature: 10 min. / 20°C
Enhancement: CuSO₄

Disclaimer:

NordiQC makes every attempt to provide accurate and up-to-date information, yet NordiQC does not make any claim or warranty regarding the accuracy of the provided information nor does it represent that the contents of the web site and protocols reflect the most recent developments in immunohistochemistry at any point in time.