

## Recommended protocol for CD117

Obtained in run 71

27 Mar 2024

### Immunostainer

Type: Leica Bond Prime

### Primary antibody

Clone: EP10  
Producer: Leica Biosystems  
Product no. / lot no.: PA0007 / 77497  
Format: Ready-To-Use (prediluted)  
Incubation time / temperature: 15 min. / 20°C

### Epitope retrieval, HIER

Device: On Board / On Machine  
Buffer: Leica Leica Bond Prime Epitope Retrieval Solution 2  
Heating time at max. temp.: 20 min.  
Maximum heating temp.: 104°C

### Visualization system

Producer: Leica  
Product / no: Bond Prime Polymer DAB Detection System / DS9284  
Linker: None  
Incubation time polymer: 8 min.  
Incubation temperature: 20°C

### Chromogen

Producer: Leica  
Product / no: Bond Prime Polymer DAB Detection System / DS9284  
Incubation time / temperature: 8 min. / 20°C  
Enhancement: None

### Disclaimer:

NordiQC makes every attempt to provide accurate and up-to-date information, yet NordiQC does not make any claim or warranty regarding the accuracy of the provided information nor does it represent that the contents of the web site and protocols reflect the most recent developments in immunohistochemistry at any point in time.