

## Recommended protocol for CK19

Obtained in run 50

14 Mar 2017

### Immunostainer

Type: Thermo Autostainer 36/48/72

### Primary antibody

Clone: RCK108  
Producer: Dako  
Product no. / lot no.: M0888 / 20015577  
Diluent: Antibody Diluent  
Dilution factor: 1:50  
Incubation time / temperature: 39 min. / 23°C

### Epitope retrieval, proteolysis

Enzyme: Protease XIV  
Enzyme producer / no: SIGMA / P5147-5G  
Incubation time / temp: 20 min. / 37°C

### Visualization system

Producer: Dako  
Product / no: EnVision FLEX / K8000/SM802  
Linker: None  
Incubation time polymer: 30 min.  
Incubation temperature: 23°C

### Chromogen

Producer: Dako  
Product / no: DAB+ / DM827 / DM823 / K800  
Incubation time / temperature: 15 min. / 23°C  
Enhancement: None

### Disclaimer:

NordiQC makes every attempt to provide accurate and up-to-date information, yet NordiQC does not make any claim or warranty regarding the accuracy of the provided information nor does it represent that the contents of the web site and protocols reflect the most recent developments in immunohistochemistry at any point in time.