

## Recommended protocol for CD8

Obtained in run 72

25 Jul 2024

### Immunostainer

Type: Leica BOND Prime

### Primary antibody

Clone: 4B11  
Producer: Leicabiosystems  
Product no. / lot no.: NCL-L-CD8-4B11 / 6110766  
Diluent: Bond Primary Antibody Diluent  
Diluent producer / no.: Leica / AR9352  
Dilution factor: 1:50  
Incubation time / temperature: 15 min. / 25°C

### Epitope retrieval, HIER

Device: On Board / On Machine  
Buffer: Leica BOND-PRIME Epitope Retrieval Solution 2  
Heating time at max. temp.: 30 min.  
Maximum heating temp.: 104°C

### Visualization system

Producer: Leica  
Product / no: BOND-PRIME Polymer DAB Detection System / DS9284  
Incubation time linker: 8 min.  
Incubation time polymer: 8 min.  
Incubation temperature: 25°C

### Chromogen

Producer: Leica  
Product / no: BOND-PRIME Polymer DAB Detection System / DS9284  
Incubation time / temperature: 10 min. / 25°C  
Enhancement: None

### Disclaimer:

NordiQC makes every attempt to provide accurate and up-to-date information, yet NordiQC does not make any claim or warranty regarding the accuracy of the provided information nor does it represent that the contents of the web site and protocols reflect the most recent developments in immunohistochemistry at any point in time.