

## Recommended protocol for ASMA

Obtained in run

14 Sep 2015

### Immunostainer

Type: Ventana Benchmark XT

### Primary antibody

Clone: 1A4  
Producer: Dako  
Product no. / lot no.: M0851 / 00045546A  
Diluent: Antibody Diluent with Casein  
Dilution factor: 1:100  
Incubation time / temperature: 28 min. / 37°C

### Epitope retrieval, HIER

Device: On Board / On Machine  
Buffer: Ventana CC1  
Heating time at max. temp.: 30 min.  
Maximum heating temp.: 95°C

### Visualization system

Producer: Ventana  
Product / no: iVIEW DAB Detection Kit / 760-091  
Incubation time polymer: min.  
Incubation temperature: 37°C

### Chromogen

Producer: Ventana  
Product / no: iVIEW DAB Detection Kit / 760-091  
Incubation time / temperature: 8 min. / 37°C  
Enhancement: None

### Disclaimer:

NordiQC makes every attempt to provide accurate and up-to-date information, yet NordiQC does not make any claim or warranty regarding the accuracy of the provided information nor does it represent that the contents of the web site and protocols reflect the most recent developments in immunohistochemistry at any point in time.