

Recommended protocol for ASMA

Obtained in run

14 Sep 2015

Immunostainer

Type: Ventana Benchmark XT

Primary antibody

Clone: 1A4
Producer: Dako
Product no. / lot no.: M0851 / 00045546A
Diluent: Antibody Diluent with Casein
Diluent producer / no.: Ventana / 760-219
Dilution factor: 1:100
Incubation time / temperature: 28 min. / 37°C

Epitope retrieval, HIER

Device: On Board / On Machine
Buffer: Ventana CC1
Heating time at max. temp.: 30 min.
Maximum heating temp.: 95°C

Visualization system

Producer: Ventana
Product / no: iVIEW DAB Detection Kit / 760-091
Incubation time polymer: min.
Incubation temperature: 37°C

Chromogen

Producer: Ventana
Product / no: iVIEW DAB Detection Kit / 760-091
Incubation time / temperature: 8 min. / 37°C
Enhancement: None

Disclaimer:

NordiQC makes every attempt to provide accurate and up-to-date information, yet NordiQC does not make any claim or warranty regarding the accuracy of the provided information nor does it represent that the contents of the web site and protocols reflect the most recent developments in immunohistochemistry at any point in time.